SELECIONE O IDIOMA

SELECT THE LANGUAGE

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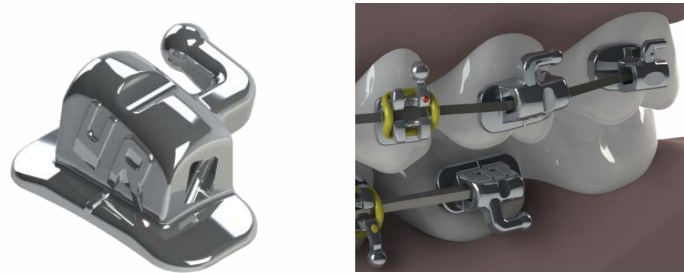
Tubes

The tubes are intended to transfer to the teeth the efforts of:

- Intra and extraoral arches;
- Springs and Elastics;
- Distalizers;
- Transpalatine bars and "Quad - Helix";
- Lip Bumpers.

They promote the correction of the positioning of the teeth during orthodontic treatment.

The tubes have the bonding and welding versions, and are available in the following models:

Simple tube

Suitable for first and second molars. It has a single slot to accommodate the intraoral arch.

Simple Convertible Tube/Simple Bracket Tube

Indicated for second premolars, first and second molars. It has a single slot to accommodate the intraoral arch. It is called "convertible" because the slot cover can be removed.

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Suitable for second molars. It has a single slot to accommodate the intraoral arch. It has as main characteristic the base displaced to mesial, to be used in partially erupted teeth.

Double Tube

Suitable for first and second molars. It has two parallel slots to accommodate the intraoral arches.

Double Convertible Tube

Suitable for first molars. It has two versions, one with auxiliary slot with orientation for cervical and another for use of Lip bumper. It is called "convertible" because the slot cover can be removed.

Triple Tube

Suitable for first molars. It has two parallel slots to accommodate intraoral arches and a centralized tube for accommodate the Facebow.

Triple Convertible Tube

Suitable for upper first molars. It has a central slot for accommodate the intraoral arch, an auxiliary slot with cervical orientation and an auxiliary tube with orientation for occlusal to accommodate the Facebow. It is called "convertible" because the slot cover can be removed.

Lingual Tube

Suitable for first molars. Attached in the lingual region, it serves as an anchorage for accessories such as the Palatal Bar and the Quad-Helix.

Use Indication:

Orthodontic Tubes are indicated to assist in the treatment of malocclusions in patients requiring orthodontic movements, which require an anchorage system. The most common clinical situations are:

- Verticalization of molars;
- Rotation of molars;
- Intrusion of anterior and molar teeth;

a) Angle's classification:

- Class I
- Class II
- Class III

b) Problems in teeth and bones:

- Prognathism;
- Retrognathism;
- Deep Overbite;
- Open bite;
- Crowding;
- Anterior crossbite;
- Posterior crossbite;
- Diastema.

Handling:

We recommend using the **"Orthodontic Tweezer for Tube"** (ref. 75.01.027)

Bonding:

We recommend using **"Orthoprimer"** (ref. 85.01.016) and **"Orthobond Plus"** (ref. 85.01.017).

Bonding Warning:

All Morelli tubes are sandblasted and undergo a thorough decontamination of the bases.

Do not touch the bonding surfaces before applying adhesive, as any contact can cause contamination, causing adhesive interface failures.

Do not apply disinfection and sterilization methods to the base as they may contaminate or impair the tubes, causing the restraint system to fail.

Do not sandblast, clean or apply other types of silane at the base of any Morelli tube, as the ideal surface treatment is already done during fabrication.

In case of base contamination, discard and replace the part.

In bonding always use orthodontic adhesives, which promote better adhesion and facilitate the removal of the residual adhesive from the tooth, preserving its enamel. Non-orthodontic resins are denser, penetrate less at the base of the brackets and are difficult to remove and consume more drills.

Proceed with adequate insulation at the time of bonding, avoiding contamination by blood or saliva.

Avoid bonding the tube with occlusion interference to prevent wear on the tooth surface or premature debonding.

Welding:

Follow the recommendations of the manufacturer of Bands.

We recommend using the "Morelli Bands" and follow its Instructions for Use (ref. 01.09.511) for a correct welding procedure.

Tube Removal:

Use removal tool for brackets and bonded tubes or instrument for band removal in the case of welded tubes.

Removing the Cover from Convertible Tubes:

Use the **"Cover Tube Detacher"** (ref. 75.01.024).

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Root resorption;
- Existing bone resorption;
- Existing Decalcification in dental enamel;
- Use in a Magnetic Resonance environment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Composition/ information about the components:

Stainless Steel

(Cr = 15% - 17,5% / Ni = 3,0% - 5,0% / Cu = 3% - 5% / Mn = Máx 1,5% / Si = Máx 0,7% / Nb = 0,15% - 0,45%)

Expiry date:

10 years

General warnings:

Orient the patient NOT TO CHEW OR BITE HARD FOOD during orthodontic treatment to prevent premature detachment.

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however, has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

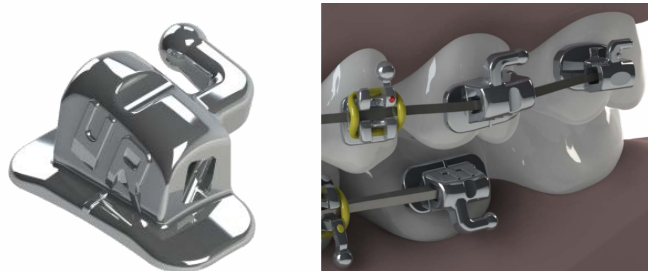
It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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