SELECIONE O IDIOMA

SELECT THE LANGUAGE

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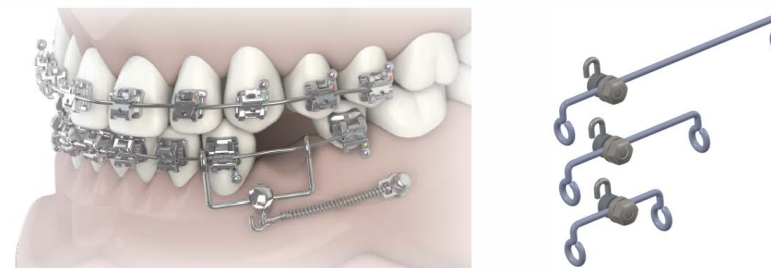
Sliding Jig for Mesialization and Distalization

It is an intraoral device, inter arches that promotes the distalization or mesialization of dental elements by applying force generated by springs or elastics skeletally anchored.

The device allows control of the force by means of a Gurin screw, which can be positioned along the arch of the cursor by modifying the intensity of the force applied to the tooth.

The Sliders are available in 3 sizes, for the right and left side of the arcades.

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a) Angle Classification:

- Class I
- Class II
- Class III

b) Problems on teeth and bones:

- Malocclusion;
- Mesialization;
- Distalization;
- Traction of teeth;
- Correction of inclinations.

Handling:

The hook is positioned along the handle with simple tightening of the screw using a specific hexagon wrench (ref. 75.02.011).

* All Sliders require the use of a hex wrench.

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic treatment;

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: gingival hyperplasia, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort and pain.

Composition / Information of components:

Stainless steel.

Validity:

10 years

Warnings:

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

This product hasn't been evaluated for compatibility and safety in Magnetic Resonance environment, the patient should be advised of this condition.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque or diseases such as gingivitis, periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

In order to protect the mucosa from contact with the Slider, if necessary, place two links of Short Chain Elastic on the rear face of the Gurin, as shown below:

ATTENTION:

Morelli Ortodontia is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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