SELECIONE O IDIOMA

SELECT THE LANGUAGE

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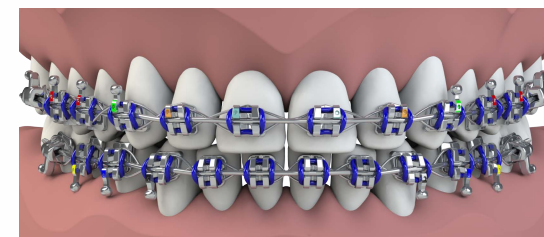
## Wire for Ligature CrNi

The Ligature wires in CrNi are auxiliary components of the orthodontic appliance intended to fix the intraoral arches to the bracket and tube slots, and in tooth ligation systems, allowing the transmission of forces that move the dental elements, resulting in corrections of malocclusions.

#### Use Indication:

The clinical situations are:

- Individual or combined bonding of several dental elements;
- Traction of dental elements;
- Retractions (active Lacebacks);
- Prevention of unwanted rotation (passive Lacebacks);
- Locking of the palatal expander after finalization.

#### Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic treatment;
- Root resorption;
- Existing bone resorption;
- Existing Decalcification in dental enamel;
- Use in a Magnetic Resonance environment.

**Side effects:**

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

**Composition/ Information of components:**

Stainless Steel: (Cr = 17,5% - 19,5% / Ni = 80% - 10,5% / Mn = Máx 2,0%).

**Expiry date:**

10 years

**General warnings:**

Do not subject the products to heat treatments.

Frequent exposure to substances with fluoride may cause the weakening of the wire.

For distal cutting in the buccal environment, make sure that the pliers retain the tip of the wire.

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however, has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended

purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

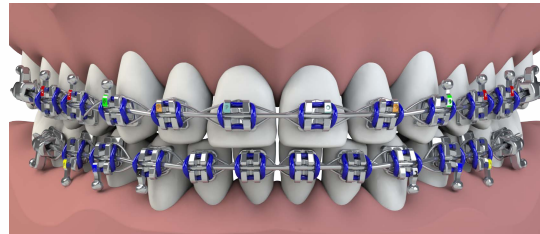
It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

**ATTENTION:**

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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