SELECIONE O IDIOMA

SELECT THE LANGUAGE

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Expansion Screw for Palatal Split

These devices are designed to correct the occlusion of certain maxillomandibular regions, usually employed in the maxillary functional orthopedics, in order to:

To widen the dental arch in its transversal direction through the lateral forces application against the maxillary posterior teeth, producing the mesopalatine suture separation, having as result, the increase in the transverse dimension of the basal bones of maxilla by means of orthopedic movements.

Such devices are fixedly employed in the patient and in conjunction with other orthodontic devices. After placement, the patient himself may be responsible for the activation of the expansion screw according to the orthodontist guidance.

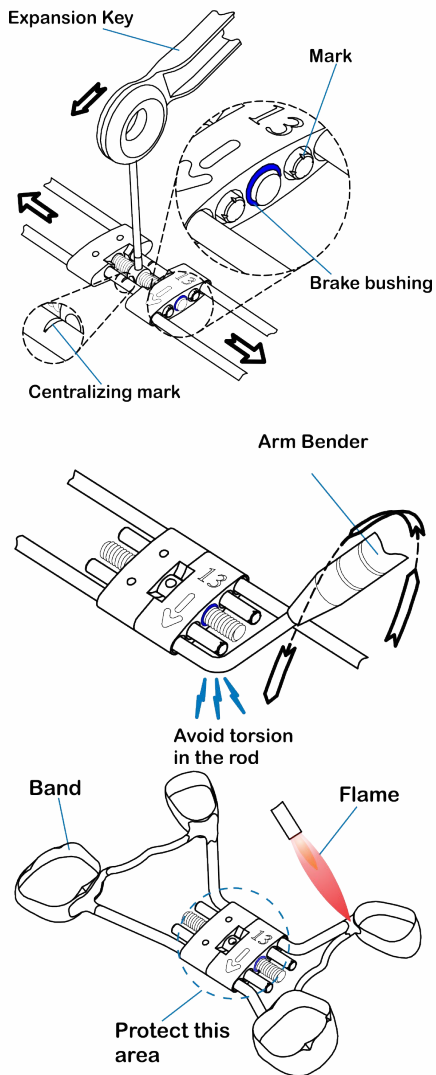
Use Indication:

The palatal disjunction expansion screws are indicated for correction of deficient maxillomandibular relations. The most common clinical situations are:

- Early mixed/permanent dentition cases;
- Postero-superior bi-lateral crossbite, as a result of the deficiency of the maxillary apical base;
- Anterior or posterior crossbite that is not too severe.

Installation:

The Morelli expansion screws has a brake bushing designed to prevent the return of the screw during its activation. This bushing is made of high quality polymer. In cases where welds are performed close to the bushing region, it is recommended to protect the bushing in order to avoid material overheating, ensuring the product effectiveness. This protection can be performed by applying specific paste, gypsum, alginate or cotton dampened in water.

When shaping the expansion screw rods use suitable tools, such as the Arm Bending for bows (cod. 75.02.001) and the trident pliers. Perform the needed folds with the desired angulation, avoiding to subject the rods to torsional stresses, which can weaken the rod as well as the weld, causing the fracture thereof.

When preparing the expansion screw avoid directing the flame on the expansion screw's body during the welding, avoiding to affect the product.

Movement:

For the movement, the appropriate key's, specifically designed for this application must be used. The main characteristic of the Expansion Screws are the activation of a central screw, using a specific key* (ref.75.01.038), which promotes the movement of metallic bodies.

* All expansion screws are accompanied by an activation key.

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Root resorption;
- Existing bone resorption;
- Existing Decalcification in dental enamel;
- Use in a Magnetic Resonance environment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any undesired side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Composition/ Information of components:

Expansion screw = Stainless Steel (Cr = Max 18% / Ni = Max 14% / Mo = Max 3% / Mn ≤ 2%)
 Activation Key (accessory) = Stainless Steel (Cr = Max 19% / Ni = MAX 10% / Mn = ≤ 2%) and Polyacetal (Polyoxymethylene = (CH₂O)_n).

Expiry date:

05 years

Warnings

The excessive rod heating may compromise the structure of the material, reducing its mechanical properties leading it to fracture.

For patient safety, in intraoral activation applications, use only the activation key supplied with the product.

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however, has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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