SELECIONE O IDIOMA

SELECT THE LANGUAGE

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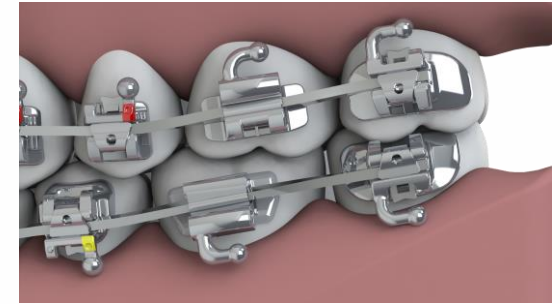
**Self Ligating Tube - SLTube**


The self ligating tubes are intended to transfer to the teeth the efforts of:

- Intraoral arches;
- Springs;
- Elastics.

**Use indication:**

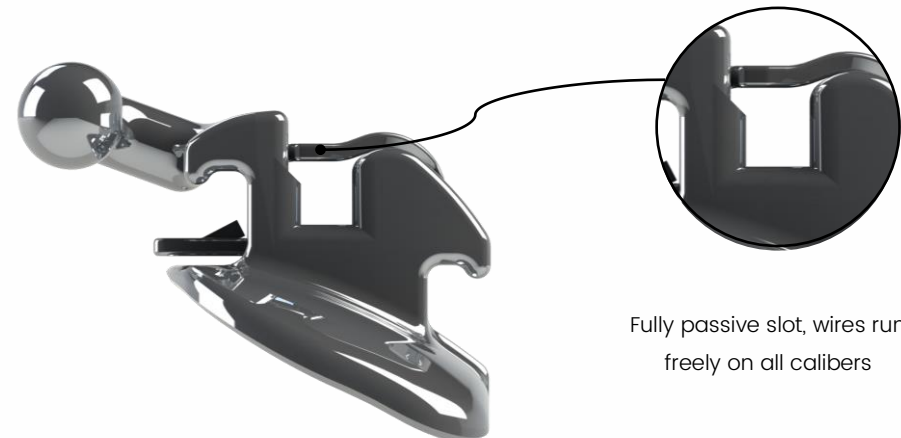Self ligating Tubes are indicated to assist in the treatment of malocclusions in patients requiring orthodontic movements. Self ligating Tubes have a passive NiTi Clip that holds the wire inside the slot and does not actively pressure the wire during treatment.

Fully passive slot, wires run freely on all calibers

The most common clinical situations are:

- Verticalization of molars;
  - Rotation of molars;
  - Intrusion of teeth.
- a) Angle's classification:
- Class I
  - Class II
  - Class III
- b) Problems in teeth and bones:
- Prognathism;
  - Retrognathism;
  - Deep Overbite;
  - Open bite;
  - Crowding;
  - Anterior crossbite;
  - Posterior crossbite;
  - Diastema.

**Handling:**

We recommend using "Tube Tweezers" (ref. 75.01.027).

**Bonding:**

For safe and effective bonding, use **Orthoprimer** (ref. 85.01.016) and **Orthobond Plus** (ref. 85.01.017).

**Bonding warnings:**

All Morelli tubes are sandblasting and undergo careful decontamination of the bases. Do not touch the bonding surfaces before applying primer / adhesive, as any contact may cause contamination, causing adhesive interface failures. Do not use disinfection and sterilization methods on the base as they may contaminate them, causing retention failure. Do not sandblast, clean or apply other types of silane at the base of any Morelli tube, as the ideal surface treatment is already done during manufacture.

In case of base contamination, discard and replace the piece.

In bonding always use orthodontic adhesives, which promote better adhesion and facilitate the removal of residual adhesive from the tooth, preserving its enamel. Non-orthodontic resins are denser, penetrate less at the base of the tube, make it difficult to remove from the tooth, and consume more drills.

Proceed with adequate insulation at the time of bonding, avoiding contamination by blood or saliva.

Avoid bonding tubes with occlusion interference to prevent tooth surface wear or premature detachment.

When bonding, remove excess adhesive to avoid clogging the clip.

**Removal:**

Use brackets removal instrument.

**Contraindications:**

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Root reabsorption;
- Existing bone reabsorption;
- Existing decalcification in dental enamel

**Side effects:**

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal

complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

**Composition/ Information of components:**

Tube: Stainless Steel Cr = 15% - 17.5% / Ni = 3.0% - 5.0%

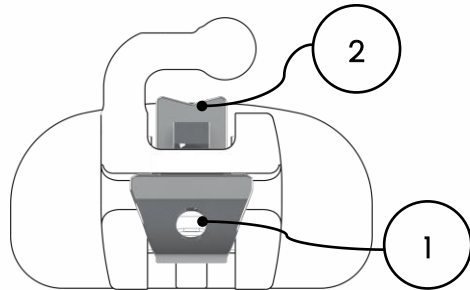
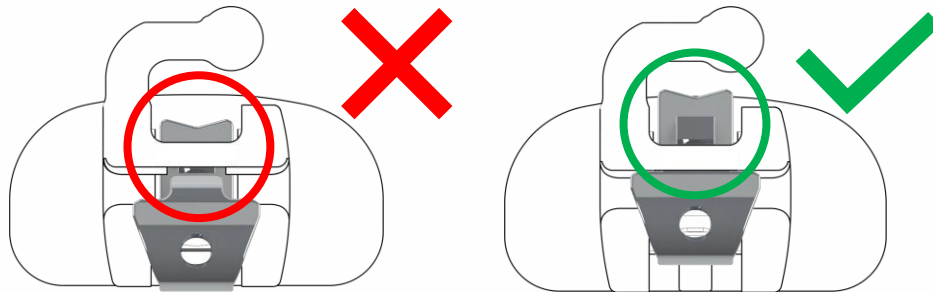
Clip: Nitinol superelastic Ni = 54.5 - 57.0%

**Expiring date:**

10 years

**Warnings:**

Close the clip, making sure to completely cover the slot, as shown.

Open the clip with an explorer or similar tip using the hole (1), and close with your finger. If there is resistance when opening, remove the tartar and use only the notch in 'V'(2), as shown.

Orient the patient NOT TO CHEW OR BITE HARD FOOD during orthodontic treatment to prevent premature detachment.

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. If allergic reactions occur, direct the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

This product did not have its biocompatibility and safety evaluated in the MRI environment, the patient should be advised about this condition.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.


It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

**ATTENTION:**

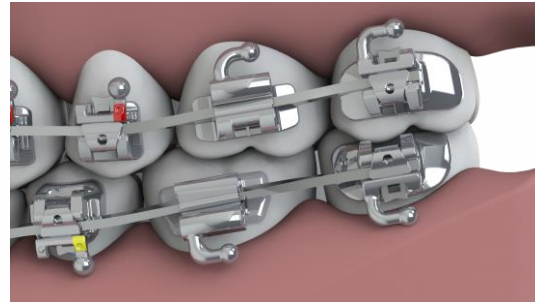Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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