SELECIONE O IDIOMA

SELECT THE LANGUAGE

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
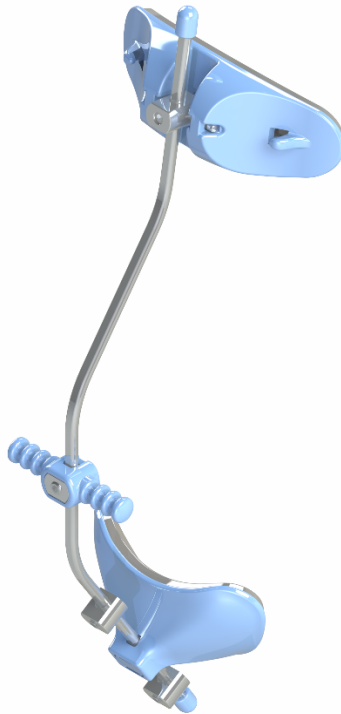
Petit Facial Mask

The Petit Facial Mask is a device for the correction of Class III malocclusion, when the main objective is the reverse traction of the maxilla. This is a mechanical, orthodontic and orthopedic procedure whose main function is the correction of the dental / skeletal problem.

Indication of use:

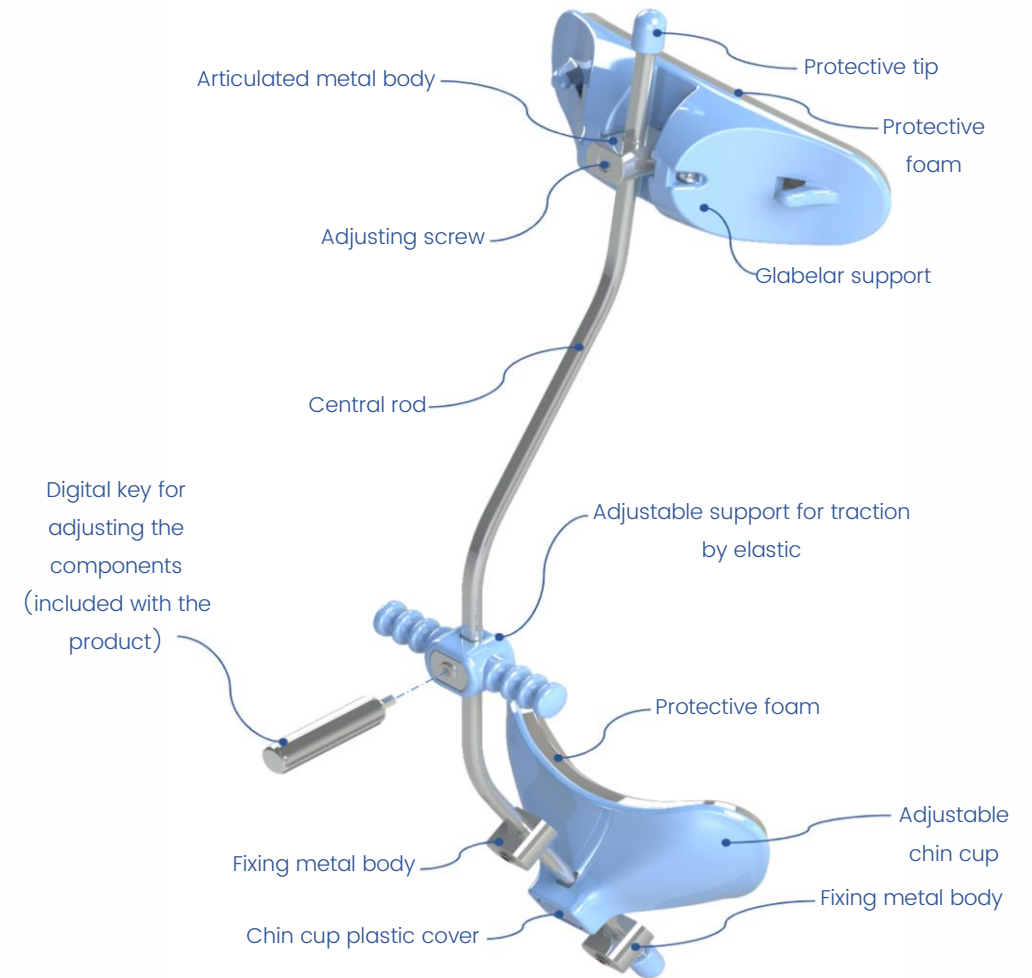- Anterior displacement of the maxilla and upper teeth;
- Hourly rotation of the mandible;
- Lingual inclination of the lower incisors;
- Increase of lower third (region of nose to ment);
- Displacement of the anterior nasal spine;
- Inhibition of mandibular growth.

Recommendations for adjusting the Petit Facial Mask

The forehead support, traction support, and chin cup may have their positions modified by simply using the "digital key" that accompanies the product. After positioning, must tighten the screws, until they remain properly secured to the central rod.

Handling:

The Petit Facial Mask can be configured in two ways:

Articulated Chin Cup

Insert the fixing metal body into the central rod.

Insert the plastic cover on the chin cup accommodation.

Insert the assembled components: chin cup + plastic cover, on the central rod.

Add the second fixing body metal in the central rod.

Make the appropriate adjustments to the patient and fix the two metal bodies through their screws.

Fixed Chin Cup


If necessary, remove the cover of the chin cup, rotating the set down on a table.

Insert the metal body into the chin cup accommodation.

Insert the assembled components: chin cup + metal body, into the central rod.


Make the appropriate adjustments to the patient and fix the metal body through its screw

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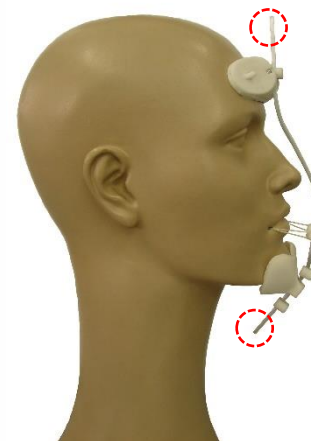After adjustments have been made, the excess tips should be removed and the protective tips replaced in order to avoid possible injuries. To cut the rod, use an abrasive disk.

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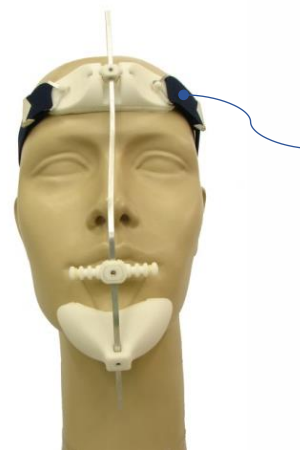In cases where it is necessary to stabilize the forehead support, there are on its sides two hooks which allow the attachment of the tape (Ref. 70.20.011 - Parietal elastic - double clasp), however it does not accompany the product.

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic / orthopedic treatment.
- Use in a Magnetic Resonance environment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Composition / Information of components:

Stainless Steel / Thermoplastic Polymer / Foam.

Expiry date:

5 years

Warnings:

It is not recommended to use the product during physical activities.

This product can cause contact dermatitis when used for people with hypersensitivity.

To promote the tightening and releasing of the screws, use only the digital key supplied with the product;

During adjustments of the adjustable support for traction, guide the patient to remain with the oral cavity closed.

The product must not be autoclaved, as this may cause damage to the plastic components;

Because it requires patient collaboration for successful treatment, use in hyperactive patients may require a longer treatment time.

Manufacturer recommends single use”

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

Recommendations to the patient:

- **Attention:** This product can cause contact dermatitis in people with hypersensitivity. In these cases, it is recommended to stop using and consult your orthodontist.
- **Skin protection:** If there is discomfort in contact with the skin, it is recommended to cover the contact surfaces with gauze or cotton pad.
- **Cleaning:** It is recommended to clean with a soft sponge, water and mild soap. Afterwards, wipe and dry in a ventilated place, sheltered from the sun.

Parietal Elastic Double Clasp

Used to provide, through low traction, better stability to the glabellar support component, present in Petit's facial masks.

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic / orthopedic treatment.
- Use in a Magnetic Resonance environment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Composition/ information of components:

Stainless Steel / Elastic Tape.

Expiry date:

5 years

Warnings:

It is not recommended to use the product during physical activities.

This product can cause contact dermatitis when used for people with hypersensitivity.

To promote the tightening and releasing of the screws, use only the digital key supplied with the product;

During adjustments of the adjustable support for traction, guide the patient to remain with the oral cavity closed.

The product must not be autoclaved, as this may cause damage to the plastic components;

Because it requires patient collaboration for successful treatment, use in hyperactive patients may require a longer treatment time.

Manufacturer recommends single use*

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

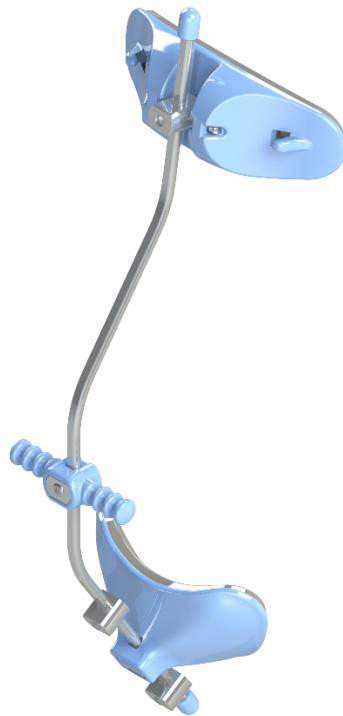
It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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