SELECIONE O IDIOMA

SELECT THE LANGUAGE

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Chin Cup – High Pool Head-Gear & Chin Cup Complete

The Chin Cup is a device used to inhibit or redirect mandibular growth by the application of orthopedic forces in the chin region. Composed of a cap with adjustable straps positioned in the parietal region, connected by traction elastics to a chin-holder.

Use Indication:

The Chin Cup is preferably indicated for the orthopedic treatment of Class III malocclusions, in cases where the patient is in the bone development phase in order to obtain a growth modification. The clinical situations are:

- Redirection of mandibular growth;
- Rotation of the mandible;
- Open bite correction;
- Control of mandibular growth.

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients unable to cooperate with treatment;
- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic treatment;
- Patients who completed their growth cycle.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: discomfort, pain and ATM dysfunction.

Composition / information on components:

Polyester, Elastic and Thermoplastic Polymer (EVA).

Expiry date:

3 years

Warnings:

This product can cause contact dermatitis when used for people with hypersensitivity.

It is not recommended to use the product during physical activities.

To attach the strips after adjusting the size and position of use desired for the patient: Sew, which is more resistant, or staple, which is more practical and can be done with paper stapler or similar, taking care not to leave tips in positions that could injure the patient (staple from the inside out). In both cases, the ends of the strips, which remain after the fastening, should be cut with fabric scissors and discarded.

The product may be indicated to the patient by a qualified professional, who should advise on the correct use of the product, foreseeing the replacement of the elastic ring regularly, as in case of ruptures, intervals of each meal or oral hygiene.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Manufacturer recommends single use.

Considering that the product is made of materials subject to recycling, its disposal must be done correctly and selectively.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

Recommendations to the patient:

- **Attention:** This product can cause contact dermatitis in people with hypersensitivity. In these cases, it is recommended to discontinue the use and consult the orthodontist.
- **Protection:** In case of skin contact discomfort, it is recommended to line the contact surfaces with gauze or cotton disc.
- **Cleaning:** It is recommended to sanitize with soft sponge, water and neutral soap. After, wipe and dry in a ventilated place, out of the sun.

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