SELECIONE O IDIOMA

SELECT THE LANGUAGE

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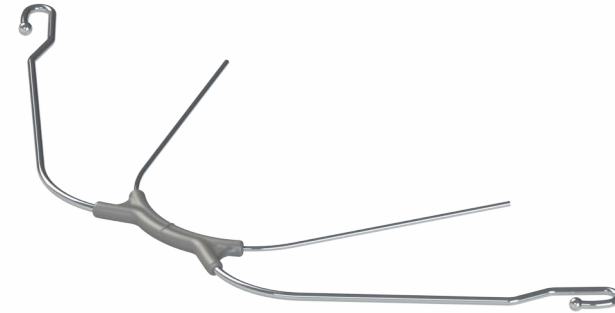
Facebows

Facebows are components of the orthodontic appliance, designed to transmit clinical forces generated outside the oral cavity in the cervical, occipital and parietal regions, which are applied in the movement and stabilization of dental elements or maxillofacial bone growth.

Use Indication:

Used with Safety Tractors, which determine the force applied to the arch, they are connected by the external arch to the several types of extraoral support, since the inner arch is connected in orthodontic tubes anchored in the teeth that will be moved or stabilized. The clinical situations are:

- Anchorage control;
- Molar distalization;
- Intrusion of molars;
- Inclination of crown from molars to distal;
- Contraction or expansion of the upper dental arch;
- Control of maxillary growth.

Installation:

The intra-buccal fitting of these products is made through a system that involves the tooth, the band, the tube and the end of the inner arch that is introduced into the molar tube.

The extraoral anchorage is made using in conjunction with Safety Tractors, which determine the force applied to the Arch, are connected by the external arch to the various types of extraoral support such as Cervical Neckpads or Headgear Adjustable Traction Interlandi.

They differ in their model dimensions, and the type of expansion screw is chosen by the professional, according to the treatment need and the morphology of the patient's dental arch.

Instructions for use to the orthodontist:

1. Avoid marks and folds in sharp edges when conforming the Facebow, to avoid fractures.
2. Round the end of the internal arcs to facilitate the coupling in the tubes.
3. Do not use elastics to anchor the Facebow: Traction Devices are essential.
4. Check the Facebow in each appointment: fractures, involuntary releases and incorrect fits can cause injuries.
5. This product is only indicated for the purpose declared by MORELLI.
6. Show clearly to the patient or responsible the correct mode of installation, give the card with **the Safety Recommendations** and inform that this recommendations **are absolutely necessary** to avoid injuries.

Safety recommendations to the patient

1. Attention at the time of placement or removal, the improper handling or fitting can cause lesions on the mouth, face and eyes. Use a mirror to assist, and always fit the bow in the tubes before coupling the traction device.
2. If the bow does not fit easily or decouple frequently inform your orthodontist. Always decouple the traction devices before remove the bow.
3. Do not use during the practice of sports or activities that can result in improper physical contact. The internal bow can cause irritations and minor temporary injuries in the mouth during the period of adaptation.
4. Wash frequently to prevent residue accumulation. If you notice cracks or corrosion points, inform to your orthodontist and request the replacement.

Storage:

When the arch is not in use, it should be stored properly in the bag designed specifically for this application.

* Accompanies unique handbag for arcs in smoked color.

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may hinder orthodontic treatment;
- Root resorption;
- Existing bone reabsorption;
- Existing decalcification in dental enamel;
- Use in a Magnetic Resonance environment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Composition/ Information of components:

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Polyamide Thermoplastic Polymer reinforced with Fiberglass.

Expiry date:

10 years

Warnings:

The product is indicated to the patient by a qualified professional, who must advise on the correct use of the product providing for the installation and removal of the Arch regularly as at intervals of each meal or oral hygiene.

This product contains Nickel and Chrome. A small percentage of the population is allergic to these metal. In case of allergic reactions, advise the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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