SELECIONE O IDIOMA

SELECT THE LANGUAGE

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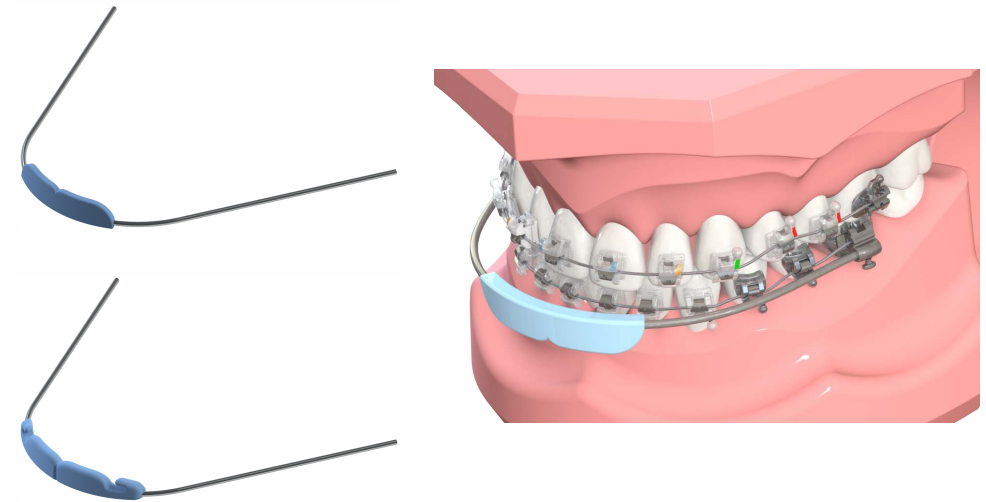
Lip Bumper

The Lip Bumper is a device intended for use in the lower arch. They are basically made up of a bow (metallic) and a lip shield (plastic), and may have hooks for anchoring.

Use Indication:

The product is connected to the orthodontic tubes installed in the lower molars that will undergo movement or stabilization, for the following clinical situations:

- Anchorage control;
- Vestibularization of the lower incisors;
- Distalization of mandibular molars;
- Intrusion of mandibular molars;
- Recovery of mandibular space;
- Inclination of molar crown to distal;
- Contraction or expansion of the lower dental arch.

Handling:**Recomendations for the patient:**

Attention when placing or removing, as improper handling can cause injury to the mouth, face or eyes. Use a mirror to assist, and always fit the arch to the tubes before attaching other components.

Inform your orthodontist if the fitting of the arch to the tube does not occur easily, or detaches frequently. Always disconnect other components before removing the arch.

Do not use while playing sports or activities that may create undue physical contact. The internal arch can cause irritation and minor temporary lesions in the mouth during the adaptation phase.

Wash frequently to prevent build-up of residue. If cracks or corrosion spots are observed, inform your orthodontist and request a replacement.

Recomendations for the professional:

Avoid marks and folds in sharp corners when forming the Lip Bumper, to avoid fractures.

Round the ends of the inner arch to facilitate coupling to the tubes.

Use pliers suitable for making omega and / or bayonet folds.

Check the Lip Bumper at each visit: fractures, involuntary loosening and incorrect fitting can cause injuries.

This product is only indicated for the purpose declared by Morelli.

Clearly demonstrate the correct installation method to the patient or guardian, give the card the safety recommendations and inform them that they are essential to avoid injuries.

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;

- Diseases or other pre-existent conditions which may hinder orthodontic treatment;
- Root resorption;
- Existing bone reabsorption;
- Existing decalcification in dental enamel;
- Use in an MRI environment;
- Patients with divergent skeletal growth (long face).

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Composition/ Information of components:

Stainless Steel: (C= Máx 0,15% / Mn= Máx 2% / Si= Máx 2% / P= Máx 0,045% / S= Máx 0,30% / Cr= Máx 19% / Ni= Máx 10% / Mo= Máx 0,6% / N= Máx 0,11%)

Lip Shield: ABS

Expiry date:

10 years

Warnings:

The product is indicated to the patient by a qualified professional, who must advise on the correct use of the product providing for the installation and removal of the Arch regularly as at intervals of each meal or oral hygiene.

Do not subject the product to heat treatments.

This product contains Nickel and Chrome. A small percentage of the population is allergic to these metal. In case of allergic reactions, advise the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non-reusable product, as its reuse can cause cross-infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

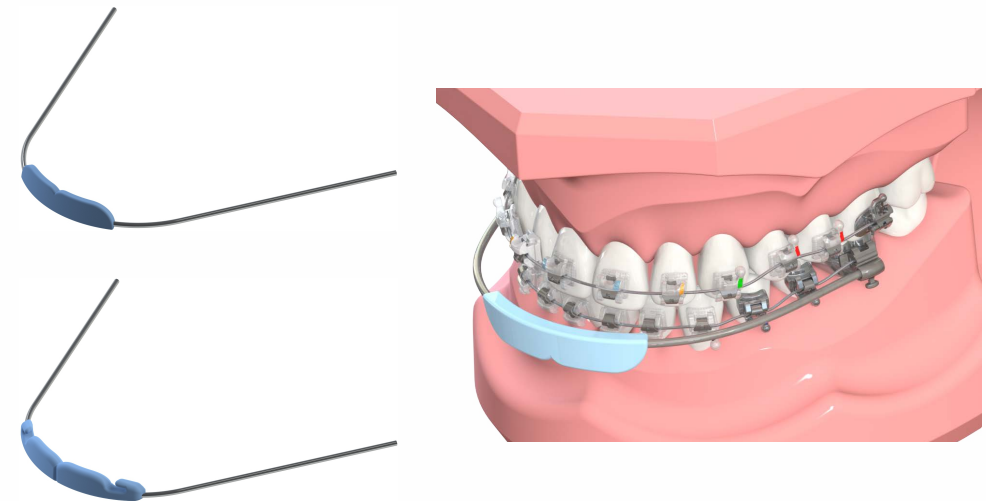
It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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