SELECIONE O IDIOMA

SELECT THE LANGUAGE

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## Lingual Buton

The Lingual Buttons are orthodontic parts designed to compose the dental apparatus, have as main characteristic:

- To be fixation of springs, ligatures and elastics for traction and tooth movement.

The lingual buttons are available in versions for welding in orthodontic bands or for direct bonding:

Welding button

Bonding button

### Use Indication:

The lingual buttons are generally used devices to promote tensile stresses. Such device, associated with others (springs, current elastics, intraoral elastic, hooks, etc.). The most common clinical situations are:

- Enables the correction of anomalies as giroversions;
- Intrusion of teeth;
- Extrusion of teeth;
- Closures of interproximal spaces (diastemas);
- Verticalization of molars.

### Handling:

We recommend using the "Orthodontic Tweezer" (ref. 75.01.022).

### Bonding:

Follow the adhesive manufacturer's recommendations.

We recommend using "Orthoprimer" (ref. 85.01.016) and "Orthobond" (ref. 85.01.015).

### Welding:

Follow the recommendations of the manufacturer of Bands.

We recommend the use of "Morelli Bands" and follow its Instructions for Use for a correct welding procedure.

### Removal:

Use tool for the removal of accessories for bonded buttons or removal tool for bands, in the case of welded buttons.

### Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral care;
- Patients incapable of cooperating with the treatment;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Existing bone resorption;
- Existing Decalcification in dental enamel.

### Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort and pain.

### Composition/ information of components:

Stainless Steel Alloy Ni = 5%

**Expiring date:**

10 years

**Warnings:**

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

For bonding always use orthodontics adhesives that promote better adherence and facilitate the removal of residual adhesive from the tooth, preserving its enamel. Non-orthodontic resins are more dense and do not properly penetrate into the base, making the removal of the product or residual adhesive more difficult, and consuming more dental burs.

Proceed with adequate insulation at the time of bonding, avoiding contamination with blood or saliva.

This product hasn't been evaluated for compatibility and safety in Magnetic Resonance environment, the patient should be advised of this condition.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque or diseases such as gingivitis, periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

**ATTENTION:**

Morelli Ortodontia is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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