SELECIONE O IDIOMA

SELECT THE LANGUAGE

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Orthobond®Plus

ORTHOBOND® PLUS is a paste-like adhesive composed of acrylic copolymers with inorganic filler, a composite, whose purpose is the fixation of orthodontic components that require bonding between part and tooth through its high adhesiveness and viscosity.

ORTHOBOND® PLUS has a unique color change system, initially appearing pink when it is expelled from the syringe. The color pink is intended to assist in positioning and / or removal of excess adhesive.

The color begins to disappear under ambient light and will disappear completely after photopolymerization. Thus, it is recommended that the components of the orthodontic appliance for bonding and receiving the adhesive at its base are applied when the patient is already properly positioned and accommodated in the chair, as well as the dental surface is pre-prepared according to guidelines below .

If application of the adhesive on the components occurs before these guidelines, the color may begin to disappear and the adhesive polymerize. However, discoloration is not an indication of total polymerization of the adhesive.

Use Indication:

Fixation of metallic, ceramic and composite components in dental enamel with orthodontic purpose.

Step by step to use the product effectively and safely

1º Step: PROPHYLAXIS / REMOVAL BIOFILM

For prophylaxis, use pumice stone diluted in water (paste point). Rub the paste with Robinson brush or rubber cup using low-angle contra-angle for biofilm removal.

After cleaning, rinse your teeth with plenty of water to remove debris and make the relative insulation of the teeth.

2º Step: DRY TEETH

Dry with oil-free or damp air. The excess moisture compromises the efficiency of the acid attack and makes it difficult to delimit the area of action of the same.

3º Step: COMPLETING THE ACID ATTACK / TOOTH PREPARATION

The acid attack / preparation varies according to the predominant surface of the tooth:

DENTAL ENAMEL: Use acid attack (phosphoric 37%), the literature recommends the time from 15 to 30 seconds, but should be extended to a longer time in cases of fluorosis, elderly people, deciduous tooth, etc.

CERAMIC: Apply the conditioner of porcelain, with spatula or micro brush, according to guidelines recommended by the manufacturer. CAUTION, the conditioner can not come into contact with soft tissue as it causes SERIOUS INJURY.

ACRYLIC RESIN / PROVISIONAL: Attack with monomer for 2 minutes, but do NOT wash afterwards.

METALLIC (AMALGAM): If the significant metal area is recommended to banding because the resin does not have good adhesion to smooth metal surfaces.

4º Step: WASH THE TEETH TO REMOVE ACID

Ensure the total removal of the acid / conditioner, removing the excess with cotton roller and following water. Guide the patient not to lick his teeth. Use a sucker because if the patient spits out excess water, the saliva will compromise the prepared area. It is recommended to use the tongue blocker or cotton roller, especially in the lower arch, to prevent the patient from touching his tongue / lips over the prepared area. From this stage the area should be kept isolated.

5º Step: DRY TEETH

Dry the teeth again with jet air free of oil or moisture. To make sure that the compressed air is free of oil, we suggest blowing the air in a mirror or polished surface. If there is any residue on the surface, the air may be contaminated, in this case check the equipment.

IMPORTANT: After drying, the enamel attacked by the acid should have a whitish appearance showing the conditioning. If this does not happen, go back to step 3.

6º Step: APPLY ORTHOPRIMER ON TEETH


Apply a drop of OrthoPrimer over the micro brush and spread this drop in up to 3 teeth. After that, repeat the operation until you have applied the OrthoPrimer over all the teeth.

The OrthoPrimer should form a film over the entire attack area. After the application remove the excess of the OrthoPrimer with air jet, in the gingival direction, avoiding to compromise the area of bonding with saliva. The use of the air jet contributes to standardize the thickness of the OrthoPrimer film.

7° Step: APPLY ORTHOPRIMER TO THE PARTS BASE


For METALLIC and CERAMIC pieces, studies show that the application of OrthoPrimer in the base increases the adhesion force by up to 20%. The application of OrthoPrimer in the base of COMPOSITE parts is obligatory (minimum time of 2 minutes). For the manipulation of the piece, use appropriate tweezers for each case (bracket tweezer, tube tweezer, etc). Apply one drop of the OrthoPrimer over the micro brush and spread this drop in up to 3 pieces.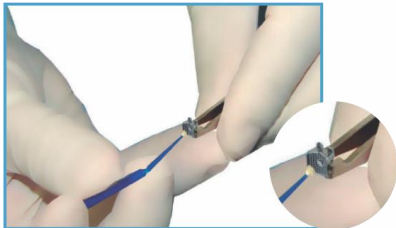

After that, repeat the operation until you have applied the OrthoPrimer on all parts bases. The OrthoPrimer should form a film over the entire base. After application remove excess OrthoPrimer with air jet, thus generating a uniform film.

8° Step: APPLY ORTHOBOND® PLUS ON BRACKETS

Compress the OrthoBond adhesive with the aid of a spatula against the entire surface of the base to eliminate bubbles, leaving the pyramid-shaped adhesive, this way prevents bubbles from forming when the piece is placed on the tooth.

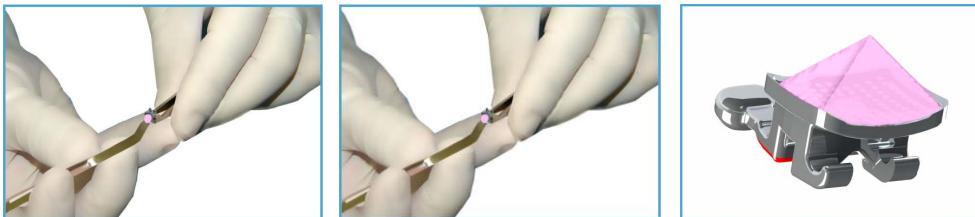
9° Step: POSITIONING THE BRACKET ON THE TOOTH

Make sure the piece is well positioned on the tooth surface using bracket positioner / star marker. Seat the bracket using the positioner blade of the tweezer.


NOTE: The positioner blade of Morelli tweezer has millimeter graduation, which assists in positioning the conference during bonding.

One indication that the amount of adhesive was sufficient is the appearance, after settling, of a light line of adhesive across the edge of the base. To remove the excess of marginal adhesive, use microbrush soaked in OrthoPrimer.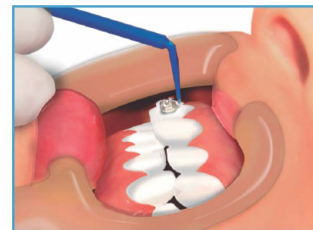

To avoid gaps in the adhesive that can cause falls or leaks, do not move the piece after the settlement. If this occurs, it is recommended to repeat steps 7 and 8.

10° Step: PHOTOPOLYMERIZING

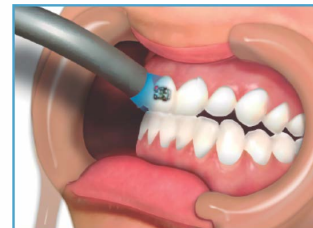Use equipment with rated current above 400 mW / cm². The tip of the photopolymerizer should be as close as possible to the base of the bracket without touching it.

The literature recommends light curing the four sides. The total time must be 40 seconds per bracket. If it is not possible to reach the four sides, this time must be distributed on the accessible sides, being the occlusal the most important.

The time may be reduced if it is used more intensively devices. It is recommended to use individual tips (photopolymerization tooth-to-tooth), if you choose multiple tip, make sure that the power for each tooth is sufficient.

Ex.: When light-curing 3 simultaneous teeth, the equipment should be at least 1200 mW / cm². After the photopolymerization, the archwire can be installed, but the total strength of the bonding will only be reached after 24 hours, so the patient should be advised to be careful in the first meals.

Removal:

After the orthodontic treatment is finished, we recommend the **Zirconia Multi Blade Burr for Resin Removal** (Ref. 75.03.001).

The zirconia burr has excellent cutting durability, resistance, no damage and no contamination of the teeth. It is equally effective in any direction of rotation, which avoids errors and increases its useful life.

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic treatment;
- Root resorption;
- Existing bone resorption;
- Existing Decalcification in dental enamel;
- Use in a Magnetic Resonance environment.
- Extensive tooth restorations;
- Cracks in the tooth;

This product is not suitable for dental restorations.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Composition/ Information of components:

Adhesive in pasty consistency, composed of acrylic copolymers with inorganic fillers.

Contains Bisphenol A.

For detailed chemical composition request the product safety sheet.

Expiry date:

2 years

Warnings:

Use gloves when handling the product.

In case of contact with skin wash off with soap and water, in case of contact with eyes wash with plenty of water. If irritation persists, seek medical attention.

In case of inhalation, direct the patient to a properly ventilated place.

If swallowed, seek medical attention.

Keep the syringe always closed when not in use to avoid drying out.

Never remove the plunger, as light and air in contact with the adhesive may damage the product.

Avoid contact and / or storage with products containing eugenol, or clove oil, as such substances are polymerization inhibitors.

This product contains acrylate monomers. A small percentage of the population is allergic to these elements. If allergic reactions, orient the patient to seek medical attention.

Proceed with adequate insulation at the time of bonding, avoiding contamination by blood or saliva.

For better effectiveness of the product, use in conjunction with the ORTHOPRIMER MORELLI.

For best product durability, store in a cool and ventilated place. Avoid exposure to high temperature or strong light. Do not freeze.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings

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