SELECIONE O IDIOMA

SELECT THE LANGUAGE

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## Composite Button

The Composite Button are orthodontic pieces designed to make part of the dental appliance, its main feature is:

- Enable the teeth traction and movement by fixation of springs, ligature wires and elastics.

### Use Indication:

The buttons are devices commonly used to promote traction forces. The most common clinical situations are:

- Enables the correction of anomalies such as rotations;
- Intrusion of teeth;
- Extrusion of teeth;
- Closings of interproximal spaces (diastema);
- Verticalization of molars.

### Bonding:

We recommend using "Orthoprimer" (ref. 85.01.016) and "Orthobond Plus" (ref. 85.01.017).

**The use of Orthoprimer in the base and in the teeth is essential for the bonding of Morelli aesthetic pieces.**

### Bonding Warning:

All Morelli accessories are sandblasted and undergo a thorough decontamination of the bases. Do not touch the bonding surfaces before applying adhesive, as any contact can cause contamination, causing adhesive interface failures.

Do not apply disinfection and sterilization methods to the base as they may contaminate or impair the accessory, causing the restraint system to fail.

Do not sandblast, clean or apply other types of silane at the base of any Morelli accessory, as the ideal surface treatment is already done during fabrication.

In case of base contamination, discard and replace the part.

In bonding always use orthodontic adhesives, which promote better adhesion and facilitate the removal of the residual adhesive from the tooth, preserving its enamel. Non-orthodontic resins are denser, penetrate less at the base and are difficult to remove and consume more drills.

Proceed with adequate insulation at the time of bonding, avoiding contamination by blood or saliva.

Avoid bonding the bracket with occlusion interference to prevent wear on the tooth surface or premature debonding.

### Debonding:

Use brackets removal instrument.

**Contraindications:**

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Root resorption;
- Existing bone resorption;
- Existing Decalcification in dental enamel;
- Use in a Magnetic Resonance environment.

**Side effects:**

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

**Expiry date:**

3 years

**Warnings:**

Accessories made of polycarbonate may have a higher debonding and breaking index, as well as natural wear in prolonged cases.

If sterilized, losses may occur in the mechanical properties of the product estimated at up to 20%.

Orient the patient NOT TO CHEW OR BITE HARD FOOD during orthodontic treatment to prevent premature detachment.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

**ATTENTION:**

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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