SELECIONE O IDIOMA

SELECT THE LANGUAGE

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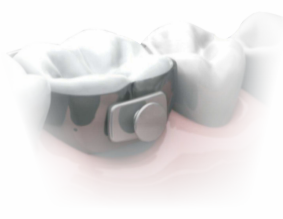
## Buton

The buttons are orthodontic parts designed to compose the dental apparatus, have as main characteristic:

- To be fixation of springs, ligatures and elastics for traction and tooth movement.

The buttons are available in versions for welding in orthodontic bands or for direct bonding:

Welding button

Bonding button

### Use Indication:

The lingual buttons are generally used devices to promote tensile stresses. Such device, associated with others (springs, current elastics, intraoral elastic, hooks, etc.). The most common clinical situations are:

- Enables the correction of anomalies as giroversions;
- Intrusion of teeth;
- Extrusion of teeth;
- Closures of interproximal spaces (diastemas);
- Verticalization of molars.

### Handling:

We recommend using the "Orthodontic Tweezer" (ref. 75.01.022).

### Welding:

Follow the recommendations of the manufacturer of Bands.

We recommend the use of "Morelli Bands" and follow its Instructions for Use for a correct welding procedure.

### Bonding:

We recommend using "Orthoprimer" (ref. 85.01.016) and "Orthobond Plus" (ref. 85.01.017).

### Bonding Warning:

All Morelli accessories are sandblasted and undergo a thorough decontamination of the bases. Do not touch the bonding surfaces before applying adhesive, as any contact can cause contamination, causing adhesive interface failures.

Do not apply disinfection and sterilization methods to the base as they may contaminate or impair the accessory, causing the restraint system to fail.

Do not sandblast, clean or apply other types of silane at the base of any Morelli accessory, as the ideal surface treatment is already done during fabrication. In case of base contamination, discard and replace the part.

In bonding always use orthodontic adhesives, which promote better adhesion and facilitate the removal of the residual adhesive from the tooth, preserving its enamel. Non-orthodontic resins are denser, penetrate less at the base and are difficult to remove and consume more drills.

Proceed with adequate insulation at the time of bonding, avoiding contamination by blood or saliva.

Avoid bonding the button with occlusion interference to prevent tooth surface wear or premature detachment.

### Debonding:

Use brackets removal instrument for bonded buttons or band removal instrument in the case of welded buttons.

**Contraindications:**

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral care;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Root resorption;
- Existing bone resorption;
- Existing Decalcification in dental enamel;
- Use in a Magnetic Resonance environment.

**Side effects:**

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

**Expiry date:**

10 years

**Warnings:**

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

Orient the patient NOT TO CHEW OR BITE HARD FOOD during orthodontic treatment to prevent premature detachment.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

**ATTENTION:**

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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