SELECIONE O IDIOMA

SELECT THE LANGUAGE

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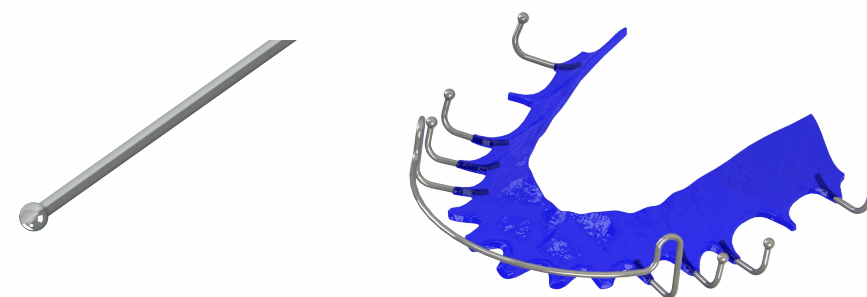
Headed Ball Clasp

The Headed Ball Clasp is a component intended for use by prosthetics in the manufacture of removable braces, both in the phase of dental retainers and for dental movements. The clasp is shaped for coupling to the teeth and the Ball Head facilitates the preparation of the braces and allows the braces to be retained to the teeth, promoting better comfort to the patients.

Use Indication:

The product is an orthodontic component in order to prevent or correct dental malocclusions, for the following clinical situations:

- Dental retainers - after orthodontic treatment of the fixed appliance;
- Dental movements - when associated with other devices, such as orthodontic expanders, for use in the growth phase, with deciduous / mixed dentition, for correcting dental malocclusions, skeletal (maxillary or mandibular) or both, for the most varied treatment of class I / II / III (cross bite, lack of space, etc.).

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may hinder orthodontic treatment;
- Root resorption;
- Existing bone reabsorption;
- Existing decalcification in dental enamel;
- Use in an MRI environment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Expiry date:

10 years

Warnings:

Do not subject the product to heat treatments.

This product contains Nickel and Chrome. A small percentage of the population is allergic to these metal. In case of allergic reactions, advise the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non-reusable product, as its reuse can cause cross-infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

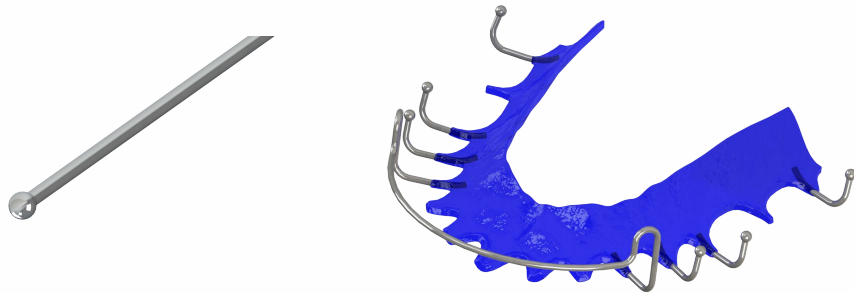
It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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