SELECIONE O IDIOMA

SELECT THE LANGUAGE

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Expansion Screw Rapid Fan Type

The Rapid Fan Expander is an orthodontic palatal expansion device, intended for anterior expansion of the upper arch and is mounted on the anterior palatine region, commonly used to treat patients with facial cranial anomaly (palate fissure).

The expander has in its posterior region, near its distal border a pin that acts as a hinge by means of elastic deformation allowing only the opening of the anterior region, keeping the posterior region united by the hinge, opening a "V" shaped angular arc.

This device is manufactured in stainless steel of high biological compatibility and has malleable rods to facilitate its conformation according to the oral morphology, fixedly applied to the patient. It is applied together with the palatal contour and its fixation is assisted by the use of lingual and buccal support wires, which allows greater rigidity of the joint after anchoring in orthodontic bands through welding.

After its total activation the expander has a system that prevents disassembly, bringing greater security to the orthodontist without harming the development of the treatment.

After placement, the patient himself may be responsible for the activation of the expander as directed by the orthodontist.

Use Indication:

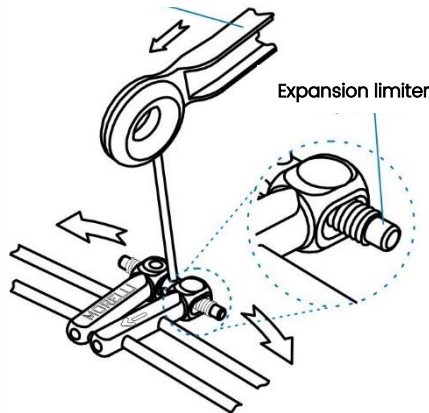
Rapid maxillary expansion is a technique widely employed with the aim of increasing the transverse dimension of the maxillary dental arch. This process preferably should occur at the

level of the apical base and with the least possible movement of the anchoring teeth within the alveolus.

This expander is intended for the increase of hypoplastic jaws of growing patients and preferably for patients with cleft lip and palate since most of these patients present specific and differentiated structural morphologies resulting in posterior crossbite and palate atresic, with an even more expressive narrowing in the anterior region of the maxilla.

Thus, the use of this type of expander produces a greater increase in the canine region, with maxillomandibular skeletal alterations more adequate to the corrections necessary to the treatment. This type of device presents better results in adolescent patients given that the formation of the palatine suture is not yet complete.

Activation key


The Rapid Fan Expander is intended for the expansion of the anterior maxillary region and has an opening limitation system to prevent dismantling after its maximum activation.

At the end of the activation the expander will emit a "click" showing its full expansion.

The materials used in manufacturing were designed to safely withstand the stresses involved in the treatment.

Avoid subjecting the expander to repeated unnecessary stresses as this may weaken the hinge, causing its fracture.

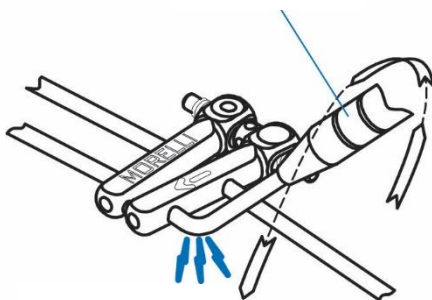Arch Bender

Avoid torsion in the rod

When framing the expander rods use suitable tools, such as the "Arch Bender" (ref. 75.02.001) and the trident plier.

Perform the necessary folds with the desired angulation, avoiding to subject the rods to torsional stresses, which can weaken the rod as well as the weld, causing them to fracture.

Activation:

For the activation, the appropriate keys, designed specifically for this application, must be used. The main characteristic of the Expansion Screw are the activation of a central screw, using a specific key*, (Ref. 75.01.038), which promotes the movement of the metallic bodies.

* All expanders are accompanied by an activation key Ref. 75.01.038.

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Root resorption;
- Existing bone resorption;
- Existing Decalcification in dental enamel;
- Use in a Magnetic Resonance environment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to the oral mucosa and tongue, difficulty speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulty maintaining oral hygiene, pain, discomfort and sensitivity.

Expiry Date:

05 years

Warnings:

The excessive rod heating may compromise the structure of the material, reducing its mechanical properties leading it to fracture.

For patient safety, in intraoral activation applications, use only the long key Ref. 75.01.038.

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however, has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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