SELECIONE O IDIOMA

SELECT THE LANGUAGE

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Expansion Screws Standard, Universal, Mini, Unilateral, Unilateral Mini and Fan

The expansion screws are complementary devices of the functional orthopedics, considered of the "slow expansion system" type, that depend on the aid of the acrylic resin specific for orthodontics, being able to be removable. These expansion screws have a plastic positioner that protects the expansion screw at the time of acrylization and contributes to the positioning of the expansion screw to the plaster model.

The screw is activated by a specific key, promoting expansion by moving the bodies.

EXPANSION SCREW STANDARD, UNIVERSAL AND MINI

They are orthodontic devices used in several types of apparatus, intended for transverse expansion of the arches from the midline thereof, and may also be longitudinally used. The screw activation results in the force application to a symmetrical opening of the bodies.

Expansion Screw Standard

Expansion Screw Universal

Expansion Screw Mini

EXPANSION SCREW FAN

They are orthodontic devices used in apparatus, intended to the anterior expansion of the superior arch, being mounted in the anterior palatine region. In the posterior region, near the distal edge of the apparatus, a flat horizontal hinge is mounted. The screw activation allows only the opening of the anterior region, keeping the posterior region united by the hinge, opening a "V" shaped angular arc.

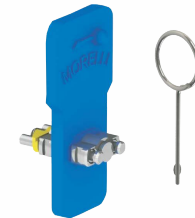
EXPANSION SCREW UNILATERAL AND UNILATERAL MINI

They are orthodontic devices used in several types of apparatuses, generally destined to the partial movement of the arcades. The screw activation of the results in the force application, using a fixed point of support.

Expansion Screw Unilateral

Expansion Screw Unilateral Mini

Use Indication:

The orthodontic expansion screws of the "slow expansion system" are indicated for the arch expansion, since they can allow the movement of a group of teeth or total. They are indicated in the majority, for patients with potential of bone growth. The most common clinical situations are:

- Increased transverse dimensions of the dental arch in both the posterior and anterior regions;
- It presents orthopedic effect and orthodontic movement;
- With the early treatment a better skeletal relation between the apical bases is achieved;
- The improvement of intercuspitation in the deciduous or mixed dentition is also achieved, providing a harmonious growth of the face;
- Correction of anterior/posterior crossbite;
- Molar distalization;
- Single tooth movement (one-sided mini expansion screw indicated);
- Anterior expansion of the upper arch (fan expansion indicated);

Installation:

They differ in their model dimensions, and the type of expansion screw is chosen by the professional, according to the treatment need and the morphology of the patient's dental arch.

Activation:

For the activation, the appropriate keys, specifically designed for this application must be used. The main characteristic of the Expansion Screw are the activation of a central screw, using a specific key*, (Ref. 75.02.026 and 75.01.038), which promotes the movement of the metallic bodies.

Ref. 75.02.026

Ref. 75.01.038

* All expanders are accompanied by an activation key Ref. 75.02.026.

Contraindications:

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Root resorption;
- Existing bone resorption;
- Existing Decalcification in dental enamel;
- Use in a Magnetic Resonance environment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to the oral mucosa and tongue, difficulty speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulty maintaining oral hygiene, pain, discomfort and sensitivity.

Expiring date:

5 years

Warnings

The excessive rod heating may compromise the structure of the material, reducing its mechanical properties leading it to fracture.

For patient safety, in intraoral activation applications, use only the long key Ref. 75.01.038.

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

This product must not be used in a Magnetic Resonance environment as it may cause interference with the images during the examination. It is up to the orthodontic practitioner to warn the patient about this contraindication and arrange for removal of the device if necessary.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however, has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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