SELECIONE O IDIOMA

SELECT THE LANGUAGE

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
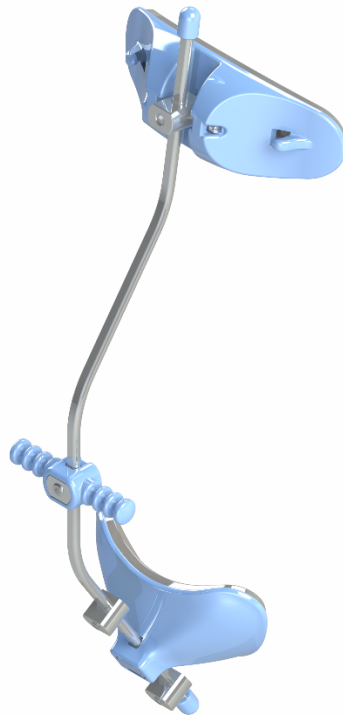
Petit Facial Mask

The Petit Facial Mask is a device for the correction of Class III malocclusion, when the main objective is the reverse traction of the maxilla. This is a mechanical, orthodontic and orthopedic procedure whose main function is the correction of the dental / skeletal problem.

Indication of use:

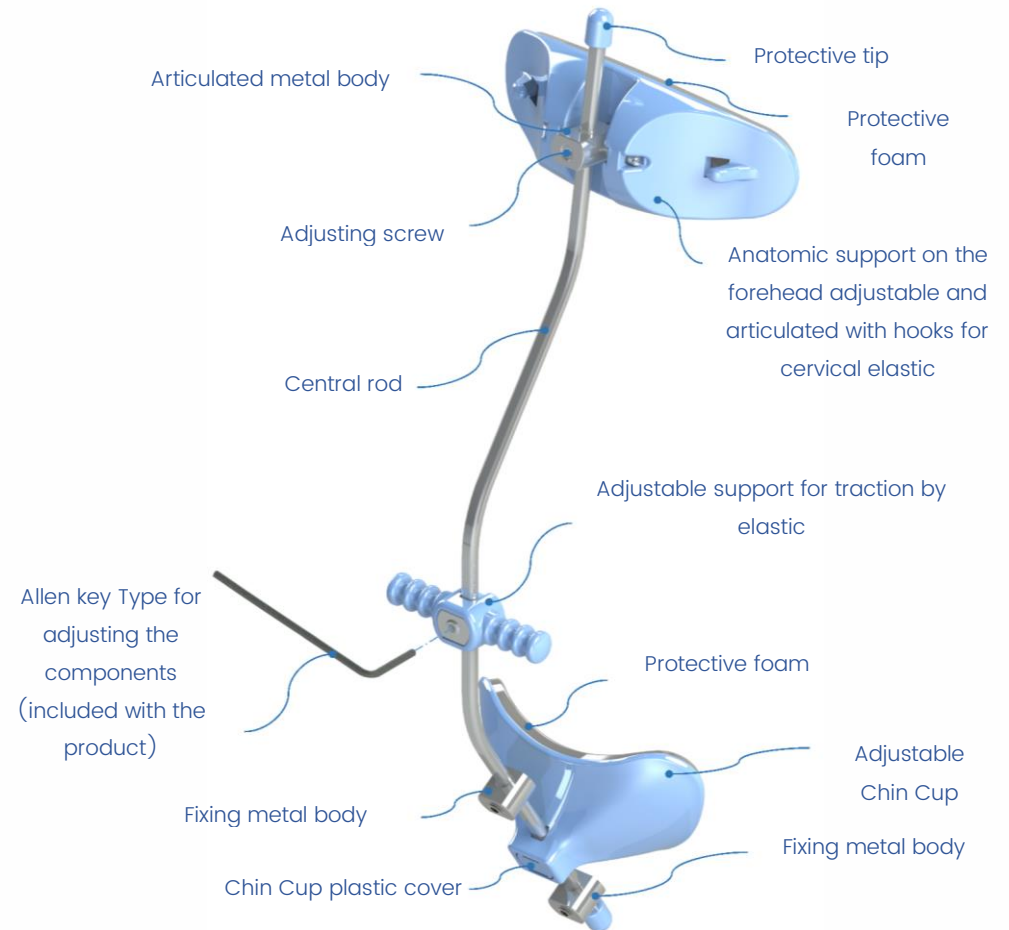- Anterior displacement of the maxilla and upper teeth;
- Hourly rotation of the mandible;
- Lingual inclination of the lower incisors;
- Increase of lower third (region of nose to ment);
- Displacement of the anterior nasal spine;
- Inhibition of mandibular growth.

Recommendations for adjusting the Petit Facial Mask

The forehead support, traction support, and chin cup may have their positions modified by simply using the "Allen key type" that accompanies the product. After positioning, must tighten the screws, until they remain properly secured to the central rod.

Handling:

The Petit Facial Mask can be configured in two ways:

Articulated Chin Cup

Insert the fixing metal body into the central rod.

Insert the plastic cover on the chin cup accommodation.

Insert the assembled components: chin cup + plastic cover, on the central rod.

Add the second fixing body metal in the central rod.

Make the appropriate adjustments to the patient and fix the two metal bodies through their screws.

Fixed Chin Cup


If necessary, remove the cover of the chin cup, through a lever movement, using the own Allen key Type.

Insert the metal body into the chin cup accommodation.

Insert the assembled components: chin cup + metal body, into the central rod.

Make the appropriate adjustments to the patient and fix the metal body through its screw.

Illustrative image of Petit Facial Mask installation

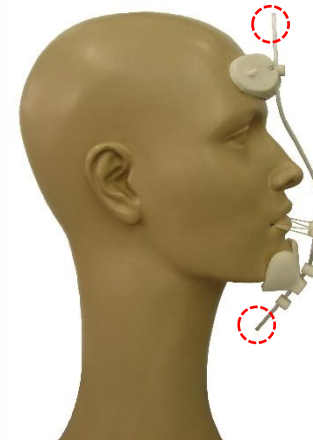After adjustments have been made, the excess tips should be removed and the protective tips replaced in order to avoid possible injuries. To cut the rod, use an abrasive disk.

Hooks for fastening the cervical elastic positioning

In cases where it is necessary to stabilize the forehead support, there are on its sides two hooks which allow the attachment of the tape (Ref. 70.20.011 – Parietal elastic – double clasp), however it does not accompany the product.

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic / orthopedic treatment;

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect. During treatment, unwanted side effects may include: tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort and pain.

Composition / Information of components:

Stainless Steel / Thermoplastic Polymer / Foam.

Validity:

5 years

Warnings:

People with hypersensitivity may experience irritation or allergy when in contact with the protective foam. In these cases, it is recommended to discontinue the use of the product.

Prolonged use of the product may result in build-up of sweat, dirt or odors. In such cases, it is recommended to perform the asepsis of the foams using a soft sponge slightly moistened with water and neutral soap. Afterwards, dry with a towel and dry in a ventilated place out of the sun. It is not recommended over-current washing water, since this procedure can damage the foam and undermine its adherence to plastic components;

To promote the tightening and releasing of the screws, use only the digital key supplied with the product;

The product must not be autoclaved, as this may cause damage to the plastic components;

Because it requires patient collaboration for successful treatment, use in hyperactive patients may require a longer treatment time.

It is not recommended to use the product during physical activities.

During adjustments of the adjustable support for traction, guide the patient to remain with the oral cavity closed.

"Manufacturer recommends single use".

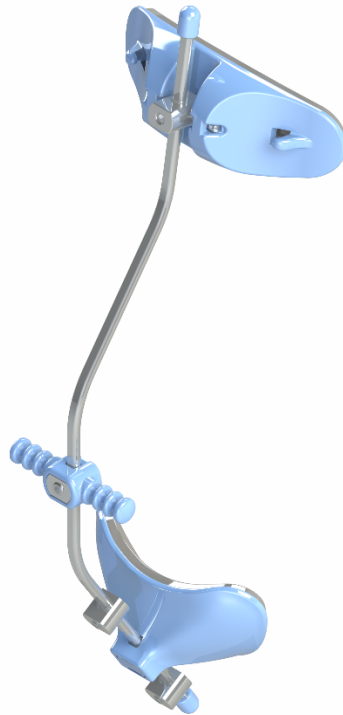
Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages.

Even the product not being used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli Ortodontia is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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