SELECIONE O IDIOMA

SELECT THE LANGUAGE

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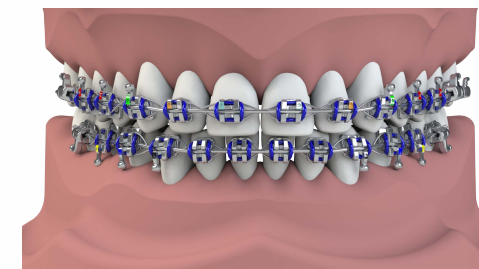
**Wire for Ligature CrNi**

The Ligature wires in CrNi are auxiliary components of the orthodontic appliance intended to fix the intraoral arches to the bracket and tube slots, and in tooth ligation systems, allowing the transmission of forces that move the dental elements, resulting in corrections of malocclusions.

**Use Indication:**

The clinical situations are:

- Fixation of intraoral arches in brackets and tubes;
- Individual or combined bonding of several dental elements;
- Retractions (active Lacebacks);
- Prevention of unwanted rotation (passive Lacebacks).

**Contraindications:**

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral care;
- Patients incapable of cooperating with the treatment;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Severe existing bone resorption;

**Side effects:**

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort and pain.

**Composition/ information about the components:**

Stainless Steel

**Expiring date:**

10 years

**Warnings:**

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

This product hasn't been evaluated for compatibility and safety in Magnetic Resonance environment, the patient should be advised of this condition.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in non-sterilized condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages.

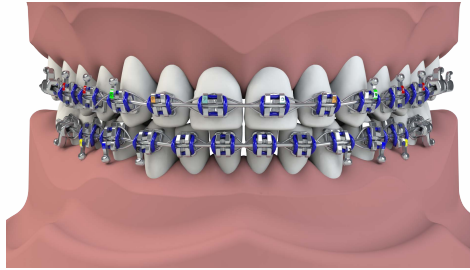
It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque or diseases such as gingivitis, periodontitis or even endocarditis.

Considering that, the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

**ATTENTION:**

Morelli Ortodontia is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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