ISELECIONE O IDIOMA

SELECT THE LANGUAGE

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Facebows

Facebows are components of the orthodontic appliance, designed to transmit clinical forces generated outside the oral cavity in the cervical, occipital and parietal regions, which are applied in the movement and stabilization of dental elements or maxillofacial bone growth.

Use Indication:

Used with Safety Tractors, which determine the force applied to the arch, they are connected by the external arch to the several types of extraoral support, since the inner arch is connected in orthodontic tubes anchored in the teeth that will be moved or stabilized. The clinical situations are:

- Anchorage control;
- Molar distalization;
- Intrusion of molars;
- Inclination of crown from molars to distal;
- Contraction or expansion of the upper dental arch;
- Control of maxillary growth.

Installation:

The intra-buccal fitting of these products is made through a system that involves the tooth, the band, the tube and the end of the inner arch that is introduced into the molar tube.

The extraoral anchorage is made using in conjunction with Safety Tractors, which determine the force applied to the Arch, are connected by the external arch to the various types of extraoral support such as Cervical Neckpads or Headgear Adjustable Traction Interlandi.

Instructions for the orthodontist:

- To conform the facebow to adapt it to each clinical case, use suitable pliers and avoid bends in living corner as they can induce the arch to fractures, as well as burrs which can harm the patient.
- Elastic for anchorage of the Facebow must not to be used, the use of Tractors is essential for the patient's safety.
- The correct way of installation of the Facebow must be clearly demonstrated to the patient and/or responsible ones.
- Arch fractures, unintentional loosening and incorrect insertion may cause injury;
- The Facebow must be checked at each appointment and problems reported during use should be reviewed by the orthodontist.
- The patient must be informed that failure to comply with the Safety Recommendations can result in injury.
- Round the ends of the internal arch to facilitate its coupling in the tubes.
- If it is necessary to promote the expansion of the arch, prefer to introduce the expansion fold after the patient is familiar with the installation of the Facebow.
- The Facebow is only indicated for the stated purpose, stated ("Indication of use") by Morelli.
- Give the patient the Safety Recommendations that comes with Facebow in the plastic bag *.

Patient Safety Recommendations:

- Be careful when inserting or removing the Facebow, improper handling or fitting may cause injury to the mucosa, cheek, face and eyes.
- Use a mirror to assist in arch placement.
- Attach the arch to the tubes before engaging the tractors.
- Detach the tractors before removing the bow from the tubes.
- Do not apply excessive forcé to the arc in the tuve, if the assembling is not happening easily, search for the orthodontist.
- Do not use the Facebow during sports or activities that may create undue physical contact.
- The internal arch of the facebow may cause irritation causing minor temporary lesions in the mouth; these are common situations and occur only in the initial adaptation phase.
- If arch disarrangement is frequent during the sleep period, inform the orthodontist so that another clinical alternative is considered.

- Wash the Facebow often in order to prevent the same from residue buildup.
- If cracks or points of corrosion are observed in the Facebow, notify the orthodontist to evaluate the need for replacement of the product.

They differ in their model dimensions, and the type of expansion screw is chosen by the professional, according to the treatment need and the morphology of the patient's dental arch.

Storage:

When the arch is not in use, it should be stored properly in the bag designed specifically for this application.

* Accompanies unique handbag for arcs in smoked color.

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral care;
- Patients incapable of cooperating with the treatment;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Existing bone resorption;
- Existing Decalcification in dental enamel.
- Patients with divergent skeletal growth (long face).

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort and pain.

Composition/ Information of components:

Stainless Steel, Nylon Thermoplastic Polymer reinforced with Fiberglass.

Expiring date:

10 years

Warnings:

The product is indicated to the patient by a qualified professional, who must advise on the correct use of the product providing for the installation and removal of the Arch regularly as at intervals of each meal or oral hygiene.

This product hasn't been evaluated for compatibility and safety in Magnetic Resonance environment, the patient should be advised of this condition.

This product contains Nickel. A small percentage of the population is allergic to these metal. In case of allergic reactions, advise the patient to seek medical attention.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque or diseases such as gingivitis, periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli Ortodontia is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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