SELECIONE O IDIOMA

SELECT THE LANGUAGE

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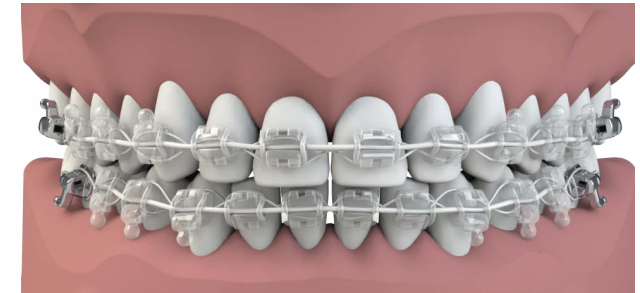
Ceramic Brackets

Bonded on the vestibular surface of the teeth, the Bracket has the main purpose of lodging the intraoral arch, both of which determine the efforts transmitted to the dental elements correcting its positioning during the dental treatment.

Use Indication:

Ceramic brackets are indicated to assist in the treatment of malocclusions. The most common clinical situations are:

a) Angle's Classification:

- Class I;
- Class II;
- Class III.

b) Problems in teeth and bones:

- Prognathism;
- Retrognathism;
- Deep Overbite;
- Open bite;
- Crowding;
- Anterior crossbite;
- Posterior crossbite;
- Diastema.

Bonding:

Follow the adhesive manufacturer's recommendations.

We recommend using "Orthoprimer" (ref. 85.01.016) and "Orthobond Plus" (ref. 85.01.017).

Perform the bonding in a similar way to the metal brackets.

Debonding:

Use brackets removal instrument. When removing, adhesive wear that surrounds the base to reduce the bond strength.

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral care;
- Patients incapable of cooperating with the treatment;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Existing bone resorption;
- Existing decalcification in dental enamel.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort and pain.

Composition/ Information of components:

Alumina Ceramics

Expiring date:

10 years

Warnings:

For bonding always use orthodontics adhesives that promote better adherence and facilitate the removal of residual adhesive from the tooth, preserving its enamel. Non-orthodontic resins are more dense and do not properly penetrate into the base, making the removal of the product or residual adhesive more difficult, and consuming more dental burs.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages

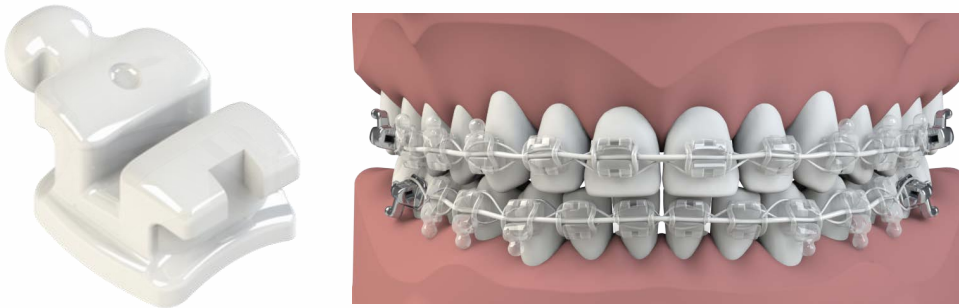
It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque or diseases such as gingivitis, periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli Ortodontia is not responsible for the results obtained by applying the product without due recautions or non-observance of the warnings.

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