SELECIONE O IDIOMA

SELECT THE LANGUAGE

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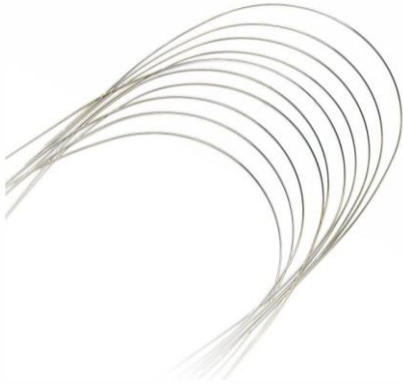
## Intraoral Archwires CrNi

Intraoral arches in CrNi are components of the orthodontic appliance, designed to transmit clinical forces generated by the interaction between the arch and brackets, promoting the movement of dental elements.

These arches are indicated for different stages of orthodontic treatment, especially in the stages where there is a greater need to control the contour of the arches.

They have good bendability, which allows execution of handles, loops or application of torques being also used in the closing of sliding spaces.

They allow the application of greater forces to the teeth presenting a low coefficient of friction in relation to the brackets, high rigidity and good weldability.

### Use Indication:

The basic objective of an intraoral arch is to achieve optimal occlusion, with well aligned arches.

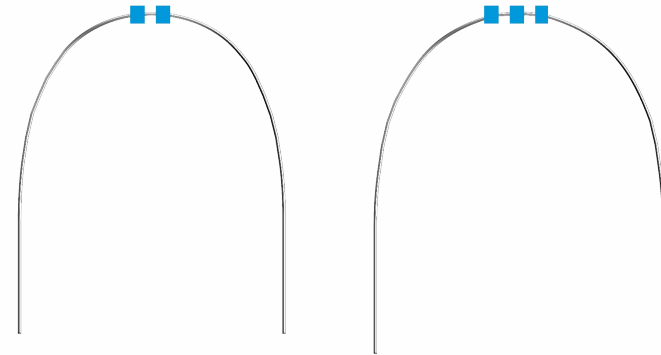
The most common clinical applications are:

- a) Angle's Classification:
  - Class I;
  - Class II;
  - Class III.
- b) Problems on teeth and bones:
  - Prognathism;
  - Retrognathism;
  - Deep bite (overbite);

- Openbite;
- Crowding (lack of spaces);
- Anterior crossbite;
- Posterior crossbite;
- Diastema.

### Identification of arches:

The arches must be chosen according to the characteristics of the clinical case, they are identified by the markings of lower arch II or upper arch III.

### Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic treatment;
- Existing bone resorption;
- Existing decalcification in dental enamel.

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It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort and pain.

**Composition/ Information of components:**

Stainless steel (Mn= 2% / Si= 1% / Cr= 18% / Ni= 9% / Mo= 0,6% / C= 0,12% / N= 0,1%)

**Expiring date:**

10 years

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Heating the arch above 200 ° C can induce fracture.

For distal cutting in the buccal environment, make sure that the pliers retain the tip of the Bow.

Excessive stresses and sharp folds can fracture the arch.

Frequent exposure to substances with fluoride may cause the weakening of the Arch.

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

This product hasn't been evaluated for compatibility and safety in Magnetic Resonance environment, the patient should be advised of this condition.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages.

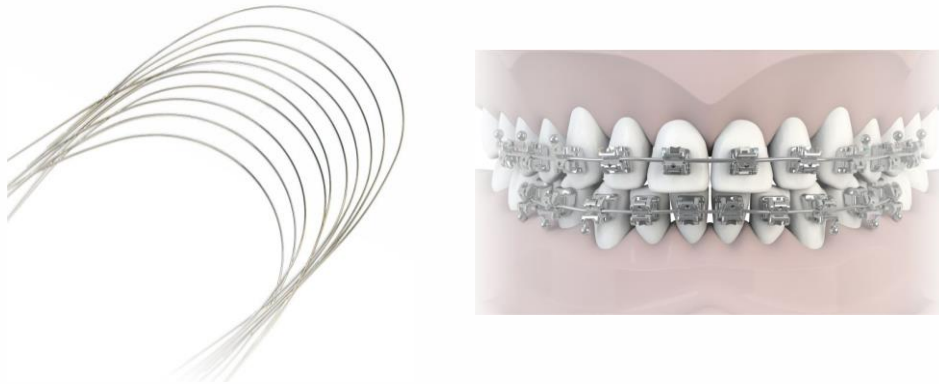
It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque or diseases such as gingivitis, periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

**ATTENTION:**

Morelli Ortodontia is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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