SELECIONE O IDIOMA

SELECT THE LANGUAGE

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Intraoral Silicone Elastics

Morelli Intraoral Silicone Elastics are devices made of biocompatible silicone rubber, designed to make up the orthodontic appliance, for the purpose of:

- Apply traction forces on dental elements.

Complementing the dental movements promoted by the other orthodontic devices applied in each clinical case.

Use Indication:

Silicone Elastics are indicated to assist in the treatment of malocclusions in patients requiring application of traction forces on dental elements. The most common clinical situations are:

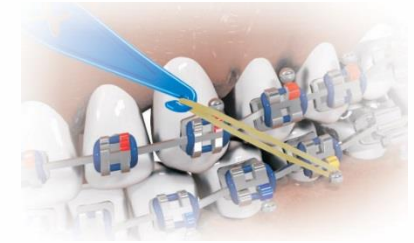
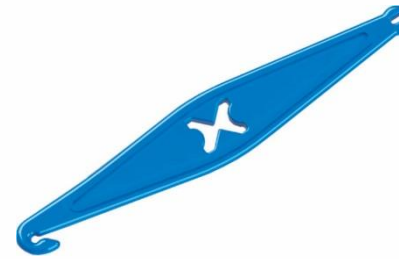
- Prevention of anteroposterior and vertical discrepancies;
- Improvement of the vertical relationship between dental elements;
- Dental asymmetries;
- Traction of retained teeth;
- Anchoring of therapeutic devices, among other applications of clinical interest during the process of correction of oral morphology.

a) Angle Classification:

- Class I
- Class II
- Class III

Installation and Removal:

We recommend using the "Elastic Positioner" (ref. 75.01.015).

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Patients hypersensitive to L-menthol (flavoring).

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: allergic reactions, discomfort and pain.

Composition / Information of components:

Silicone Platinum Rubber

Expiring Date:

3 years

Warnings:

The product is indicated to the patient by a qualified professional, who should advise on the correct use of the product, foreseeing the replacement of the elastic ring regularly, as in case of ruptures, intervals of each meal or oral hygiene.

Product slightly flavored with mint essence. A small percentage of the population is allergic to menthol. If allergic reactions occur, advise the patient to remove the elastic and seek medical advice.

Store in a cool and airy place. Avoid exposure to high temperature or intense light.

The product follows the dimensional patterns, initial strength (F_0) and residual after 24h (F_{24}), as established in the norms ABNT NBR ISO 21606 and DIN EN ISO 21606.

Keep the product in its original packaging and always closed in order to avoid drying.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

The product is supplied clean, free of dirt, but in unsterile condition. Since the packaging has not been violated, are not necessary cleaning measures, disinfection or sterilization. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages.

In order to maximize the performance of the product and extend its useful life, it is recommended to activate the elastic ring prior to application. To do this, simply stretch and relax the elastic ring 3 to 4 times.

Persons with motor difficulties or early age should be assisted by a responsible adult, which is properly supervised by a qualified professional in case of replacement of the elastic rings.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque or diseases such as gingivitis, periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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