SELECIONE O IDIOMA

SELECT THE LANGUAGE

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Composite Lingual Button

The Composite Lingual Button are orthodontic pieces designed to make part of the dental appliance, its main feature is:

- Enable the teeth traction and movement by fixation of springs, ligature wires and elastics.

Use Indication:

The lingual buttons are devices commonly used to promote traction forces. The most common clinical situations are:

- Enables the correction of anomalies such as rotations;
- Intrusion of teeth;
- Extrusion of teeth;
- Closings of interproximal spaces (diastema);
- Verticalization of molars.

Bonding:

Follow the adhesive manufacturer's recommendations.

We recommend using "Orthoprimer" (ref. 85.01.016) and "Orthobond" (ref. 85.01.015).

Debonding:

Use an instrument for removal of accessories.

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;

- Allergy to any of the materials of the orthodontic appliance;
- Diseases or other pre-existing conditions that may hinder orthodontic treatment;
- Existing bone resorption;
- Existing decalcification in dental enamel.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort and pain.

Composition / Information of components:

Polycarbonate reinforced with 30% glass fiber.

Validity:

3 years

Warnings:

Accessories made of polycarbonate may have a higher debonding and breaking index, as well as natural wear in prolonged cases.

For bonding always use orthodontics adhesives that promote better adherence and facilitate the removal of residual adhesive from the tooth, preserving its enamel. Non-orthodontic resins are more dense and do not properly penetrate into the base, making the removal of the product or residual adhesive more difficult, and consuming more dental burs.

If sterilized, losses may occur in the mechanical properties of the product estimated at up to 20%.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element or even irreversible damages.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque or diseases such as gingivitis, periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli Ortodontia is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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