SELECIONE O IDIOMA

SELECT THE LANGUAGE

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Latex Elastics

The Latex elastics are manufactured in natural latex designed to compose the orthodontic appliance, in order to:

- Apply traction forces on dental elements.

Complementing the dental movements promoted by the other orthodontic devices applied in each clinical case.

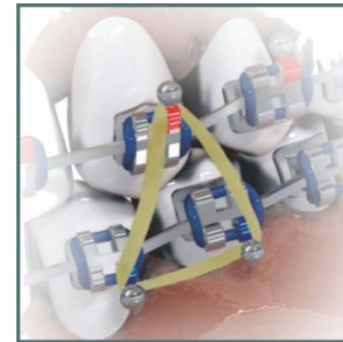
Use Indication:

Latex Elastics are indicated to assist in the treatment of malocclusions in patients requiring application of traction forces on dental elements. The most common clinical situations are:

- Prevention of anteroposterior and vertical discrepancies;
- Improvement of the vertical relationship between dental elements;
- Dental asymmetries;
- Traction of retained teeth;
- Anchoring of therapeutic devices, among other applications of clinical interest during the process of correction of oral morphology.


a) Angle Classification:

- Class I
- Class II
- Class III

Installation and Removal:

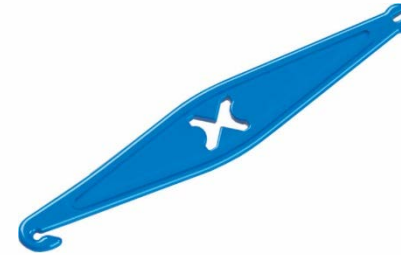We recommend using the "Elastic Positioner" (ref. 75.01.015).

Contraindications:

Morelli Ortodontia supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Patients hypersensitive to natural latex;
- Patients hypersensitive to L-menthol (flavoring);
- Diseases or other pre-existing conditions that may hinder orthodontic treatment.

Side effects:

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal

complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

Composition / Information of components:

Natural Latex Rubber

Expiration date:

3 years

Warnings:

CONTAINS NATURAL LATEX. MAY CAUSE ALLERGY. A small percentage of the population is allergic to Natural Latex Rubber. If allergic reactions occur, advise the patient to remove the elastic and seek medical advice.

The product is made from natural latex rubber and the release of proteins can cause allergy or anaphylactic response in patients. Make sure the patient is not hypersensitive to latex proteins prior to application of the product.

Product slightly flavored with mint essence. A small percentage of the population is allergic to menthol. If allergic reactions occur, advise the patient to remove the elastic and seek medical advice.

The product is not indicated for patients with spina bifida. About 50% of people with spina bifida develop latex allergy due to repeated exposure of latex products used in medical or surgical procedures.

Store in a cool and airy place. Avoid exposure to high temperature or intense light, in order to avoid the decrease of the useful life of the product and loss of the mechanical properties.

The product follows the dimensional patterns, initial strength (F_0) and residual after 24h (F_{24}), as established in the norms ABNT NBR ISO 21606 and DIN EN ISO 21606.

Persons with motor difficulties or early age should be assisted by a responsible adult, which is properly supervised by a qualified professional in case of replacement of the elastic rings.

Keep the product in its original packaging and always closed in order to avoid drying.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied clean and free from dirt, in unsterile condition, not being necessary cleaning, disinfection or sterilization measures. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients about correct use of the product, providing for regular replacement in the case of rupture, interval between meals and the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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